

Regulatory information

The mobile stand is for use exclusively with the pneumatic tourniquets D & D.

He's delivered with :

- A mounting plate specially designed for pneumatic tourniquets D & D
- An arm equipped with a handle to facilitate movement.
- A walk with 4 braked castors
- A storage basket for the pneumatic armbands and extension cords.

1 – Device identification

Mobile stand



2 - References


Reference	Designation	Dimensions in m		Weight in kg
		Height	Footprint	
A10701	Mobile stand for pneumatic tourniquet	1	Ø 0.56	14

3 - Instructions for use

- The mobile stand is used with pneumatic tourniquets D & D.
- The pneumatic tourniquet is maintained on the mounting plate by screws.
- To prevent risk of device falling, do not propel the unit mounted on mobile stand, a handle is provided to make any manoeuvre secure by pulling or pushing the device to cross in front of any obstacles.
- The moving is done by pushing forward. Keep one hand on the handle in case of uneven ground.
- To prevent inadvertent movement, it is strongly recommended to lock the wheels brakes.
- Any movement of the device must be disconnected for the power supply.
- Separate the electrical power cord to the castors.

4 - Identification and characteristics of differents subsets

Identifier	Designation	Characteristics
A	Gallows with platinum	Steel + epoxy paint
B	Basket	Rilsan-coated steel
C	Base with 5 branches	Steel + shockproof plastic coating
D	Castors	Clevis and high quality nylon wheel. Polypropylene wheel body, gray thermoplastic rubber bandage

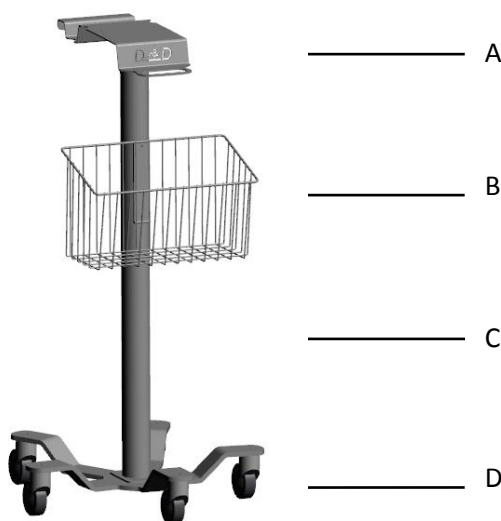

5 - Cleaning and disinfection

Spray disinfectant solutions.

6 -Maintenance

Cleaning rollers to remove any residual dust or materials which might interfere with the movement of the apparatus.

7 - Packaging

Reference	Packaging unit
A10701	1

8 - Storage

Type of packaging	Storage area	Temperature	Humidity	Atmospheric pressure
Original packaging	Ventilated area	-10° à 40 ° c	30 to 40 %	500 to 1060 hpA

9 - Warrentee

This warranty provides assurance for the customer who purchases a D & D product that should the product fail to function to D & D published specifications during the term of this warranty, will either replace or repair.

The guarantee period is 2 (two) years from the date of purchase.

The product must be used in accordance with its labeling and may not be altered or subjected to misuse, abuse, accident or improper handling.